


**INSTRUCTIONS FOR USE:**

Application:

1. Select pre-formed core build-up cap.
2. Fill the pre-formed cap with light curing or self curing composite.
3. Place cap over tooth and root canal post.
4. After curing the composite, remove pre-formed cap.
5. Finish the preparation in the cervical and occlusal area.

Core Build-up Caps (sizes in mm):

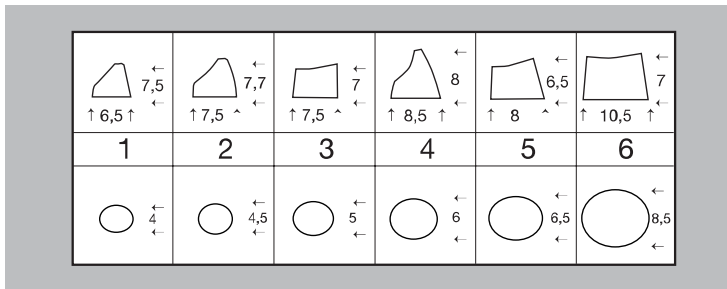

Refills:

Size 1 – Anterior small

Size 4 – Canine large

Size 2 – Anterior med.

Size 5 – Molar small/med.

Size 3 – Premolar

Size 6 – Molar large

Non Sterile! For single use!

If light curing material is used, cure with double exposure time!