Fini™ Polishing Paste Instructions for Use

To reduce waste and minimize splatter, place a small amount of paste on a weighted (stationary) mixing slab. Dab the brush or disk into the paste while rotating at approximately 10,000 RPM's uniformly incorporating it into the fibers.

Use brush or disks at low speeds (20,000 to 30,000 RPM's) with light intermittent strokes to minimize heat. Reapply paste as necessary. Prior to completion, use feather-like strokes at 30,000 RPM's.

Note: Brushes can be autoclaved up to 280 degrees F.

Store at room temperature 74°F/23°C.

Limited Warranty

Express Limited Warranty: *Pentron Clinical Technologies, LLC* warrants that this product is not defective for a period of one year.

Buyer's Exclusive Remedy: In the event that this product proves to be defective within a period of one (1) year from the date of shipment to Buyer, Buyer's exclusive remedy shall be at *Pentron Clinical Technologies, LLC's* option either replacement of the product or defective part or component, free of charge, or refund of the purchase price. Buyer is only entitled to this remedy upon showing that the product has been stored and used in accordance with *Pentron Clinical Technologies, LLC's* instructions, with the exception that this warranty nevertheless extends to products whose expiration dates fall within the one-year limited warranty period.

Disclaimer of Consequential Damages: *Pentron Clinical Technologies, LLC* hereby disclaims any liability for loss or damage, direct or consequential, that result from the use of this product.

Disclaimer of All Other Warranties: THERE ARE NO OTHER WARRANTIES OF ANY KIND, EXPRESS OR IMPLIED, CONNECTED WITH THE SALE OF THIS PRODUCT. *Pentron Clinical Technologies, LLC* SPECIFICALLY DISCLAIMS ANY IMPLIED WARRANTY OF MERCHANTABILITY.

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