

INSTRUCTIONS FOR USE:

Cervical Sensitivity

1. Providing the tooth is not too sensitive, clean with pumice and rinse with water.
2. Apply PrepEze® Desensitizer for 30 seconds with a cotton pellet, or microbrush using gentle but firm rubbing motion.
3. Blot or let air dry with a cotton pellet and apply 2 more coats of desensitizer for a total of 3 coats. Do not rinse.
NOTE: Pumice is not necessary for the desensitizer to be effective.

Under Amalgams

1. Prep tooth as normal. Leave the tooth slightly moist and apply *PrepEze* Desensitizer for 30 seconds with a cotton pellet, or microbrush, using gentle but firm rubbing motion.
2. Blot or let air dry between each coat with a cotton pellet.
3. Apply 2-3 coats of desensitizer. Let dry.
4. Proceed to place your amalgam. If a bonding agent is being used beneath the amalgam, apply the desensitizer after the removal of the smear layer.

Veneers, Inlays, Onlays, Crowns

NOTE: At temporary seating apply desensitizer prior to placement of the temporary cement.

1. Prep tooth as normal, apply *PrepEze* Desensitizer for 30 seconds with a cotton pellet, or microbrush, using gentle but firm rubbing motion.
2. Blot or let air dry between each coat with a cotton pellet.
3. Apply 2-3 coats of desensitizer. Let dry.

NOTE: At final seating place *PrepEze* Desensitizer on tooth after the smear layer has been removed (usually accomplished by applying an acid etchant contained in the bonding system). *PrepEze* Desensitizer does not require smear layer removal in order to desensitize. Repeat the same application method as explained for the temporary seating in steps 1-3 above.

Under All Composite Restorations

1. Prep tooth. Apply etchant to remove smear layer and rinse
2. Place several drops of *PrepEze* Desensitizer in dappen dish. Moisten a cotton pellet or microbrush with the desensitizer. With a gentle but firm rubbing motion, apply for 20-30 seconds.
3. Air dry or blot with a cotton pellet or let air dry between each coat.
4. Apply 2-3 coats. Let dry.

In Conjunction With Glass Ionomer Cements

1. Prep tooth as normal. Apply *PrepEze* Desensitizer for 30 seconds with a cotton pellet, or microbrush, using gentle but firm rubbing motion.
2. Blot or let air dry between each coat with a cotton pellet.
3. Apply 2-3 coats of desensitizer. Let dry.
4. Place the glass ionomer material. Leave tooth moist with *PrepEze* Desensitizer when seating the restoration.

Before and After Tooth Whitening

1. Place several drops of *PrepEze* Desensitizer in dappen dish. Moisten a cotton pellet or microbrush with the material and with a gentle but firm rubbing motion -apply for 20-30 seconds.
2. Let air dry or blot with a cotton pellet or let air dry between each coat.
3. Apply 2-3 coats. Let dry.

Prior to Scaling

1. Place several drops of *PrepEze* Desensitizer in dappen dish. Moisten a cotton pellet or microbrush with the material and with a gentle but firm rubbing motion -apply for 20-30 seconds.
2. Let air dry or blot with a cotton pellet.
3. Apply 2-3 coats. Let dry.
4. Scale teeth.
5. Re-apply *PrepEze* Desensitizer - 2-3 coats.
6. Let air dry or blot.

Pentron and PrepEze are registered trademarks of Pentron Clinical Technologies, LLC.

