

English

INSTRUCTIONS FOR USE:

Protect-It[®] COMPOSITE SURFACE SEALANT

1. Finish and polish composite restoration. Rinse and dry. If sealing an older restoration, pumice, then rinse and dry.
2. Apply supplied 37% phosphoric acid etchant to composite surfaces and extend out to cover approximately 2mm of enamel adjacent to restoration margin. Wait 20 seconds and rinse thoroughly to remove etchant.
3. Apply *Protect-It* Surface Sealant with applicator provided. Be sure to cover entire restoration surface and the marginal area. Blow thin with a jet of air.
4. Light cure *Protect-It* Surface Sealant for 20 seconds.
5. Minimal or no finishing or polishing is necessary.

