

**INSTRUCTIONS FOR USE:****TempSpan® Glaze**

TempSpan Glaze is formulated with a nano-particulate filled resin that eliminates not only the time consuming step of polishing provisional restorations, but also the oxygen inhibited smear layer. One application of *TempSpan* Glaze effectively coats the provisional restoration and fills in any surface voids. After light cure, *TempSpan* Glaze creates an immediate lustrous gloss for beautiful, natural-looking provisionals. In addition to creating a fast and lustrous shine, *TempSpan* Glaze contributes to increased resistance to surface wear and staining.

**Indication:**

- Glazing of provisional restorations.

**Contraindication:**

- Not indicated for glazing definitive composite restorations.

Finishing and Polishing the Provisional

1. Using either carbide or diamond finishing burs contour the provisional restoration.
2. Rinse the provisional with water to remove any loose debris. Dry thoroughly.
3. Apply a thin coat of *TempSpan* Glaze to the provisional with an applicator brush.
4. Optional step: Lightly air thin *TempSpan* Glaze to ensure even coverage.
5. Light-cure each surface of the provisional for thirty seconds. Apply additional coats as desired. Note: *TempSpan* Glaze has been formulated to cure with both halogen and LED curing lights.
6. Cement the provisional restoration. Pentron Clinical recommends using *TempSpan* CMT Temporary Cement for cementation.

**TempSpan Glaze****REF N69G**

All refills include:

- 1 – 6ml bottle of *TempSpan* Glaze
- Instructions and MSDS

The full *TempSpan* Provisional System includes:

*TempSpan* Dual Cure Temporary Crown & Bridge Material

Available in the following shades:

|           |                  |             |                  |
|-----------|------------------|-------------|------------------|
| <b>A0</b> | <b>REF N69AA</b> | <b>A3.5</b> | <b>REF N69AE</b> |
| <b>A1</b> | <b>REF N69AB</b> | <b>B1</b>   | <b>REF N69AF</b> |
| <b>A2</b> | <b>REF N69AC</b> | <b>C2</b>   | <b>REF N69AG</b> |
| <b>A3</b> | <b>REF N69AD</b> |             |                  |

All refills include:

- 1 - 50ml/80gm cartridge of *TempSpan* Dual-Cure Temporary C&B Material
- 15 Mixing Tips
- Instructions and MSDS

**TempSpan CMT Temporary Cement****REF N69CA**

All refills include:

- 1 - 4.5ml/6.9gm auto-mix syringe of *TempSpan* CMT Temporary Cement
- 12 Mixing Tips
- Instructions and MSDS

**TempSpan Clear Matrix Material****REF Q34CB**

All refills include:

- 4 - 50ml/52gm cartridges of *TempSpan* Clear Matrix Material
- 16 Mixing Tips
- Instructions, MSDS, and Batch Report