

**INSTRUCTIONS FOR USE:****TempSpan® Clear Matrix Material**

*TempSpan* Clear Matrix Material is a clear pre-operative impression material that will reproduce the finest detail resulting in extremely accurate provisional restorations. Use as your preliminary impression material for intra-oral and extra-oral polymerization of dual-cure provisional materials. Light curing dual-cure provisional materials may eliminate or drastically reduce the oxygen-inhibited smear layer.

**Indications:**

- Use for intra-oral polymerization of light activated provisional materials.
- Use as a general impression material.

**Contraindications:**

- Do not use as a bite registration material.

**Creating the clear matrix**

1. Select a rigid, clear plastic tray sized to provide a 2-3mm thickness of clear matrix material.
2. Brush a thin layer of *Correct VPS®* Tray Adhesive onto the tray and allow it to air dry for five minutes. Tray adhesive may be applied up to 30 minutes prior to the application of clear matrix material.
3. Inject *TempSpan* Clear Matrix Material into the tray. The working time is 1 minute.
4. Seat tray.
5. Remove the tray and impression after two minutes in the mouth. Rinse the impression under cold water and blow dry.
6. Prepare the tooth or teeth as needed.
7. Take the final impression; *Correct Plus®* Hydrophilic Impression Material is recommended. *TempSpan* Clear Matrix Material may be used to take the final impression by relining it with *Correct Plus* Thick n' Thin® or *Correct Plus* Light Body.

For instructions on creating the provisional restoration please refer to the *TempSpan* Dual-Cure Temporary Crown & Bridge Material instructions.

**TempSpan Clear Matrix Material****REF Q34CB**

All refills include:

- 4 - 50ml/ 52gm cartridges of *TempSpan* Clear Matrix Material
- 16 Mixing Tips
- Instructions, MSDS, and Batch Report

**The full TempSpan Provisional System also includes:****TempSpan Dual Cure Temporary Crown & Bridge Material**

Available in the following shades:

<b>A0</b>	<b>REF N69AA</b>	<b>A3.5</b>	<b>REF N69AE</b>
<b>A1</b>	<b>REF N69AB</b>	<b>B1</b>	<b>REF N69AF</b>
<b>A2</b>	<b>REF N69AC</b>	<b>C2</b>	<b>REF N69AG</b>
<b>A3</b>	<b>REF N69AD</b>		

All refills include:

- 1 - 50ml/80gm cartridge of *TempSpan* Dual-Cure Temporary C&B Material
- 15 Mixing Tips
- Instructions, and MSDS

**TempSpan CMT Temporary Cement****REF N69CA**

All refills include:

- 1 - 4.5ml/6.9gm auto-mix syringe of *TempSpan* CMT Temporary Cement
- 12 Mixing Tips
- Instructions, and MSDS

**TempSpan Glaze****REF N69G**

All refills include:

- 1 - 6ml bottle of *TempSpan* Glaze
- Instructions, and MSDS