

Polish for Zirconia, Lithium Disilicate,  
Porcelain and Composite.

# Amber<sup>®</sup> Glow

## *Polishing Paste*



*Let your restorations glow!*

- ✓ All glass-ceramics
- ✓ Zirconia
- ✓ Hybrid resin-ceramics
- ✓ Composite restorations

Recommended rpm : 10,000 rpm  
Size : 10 g/jar

**Instructions:** For best results, rubber wheel the area to be polished and Amber Glow with a felt wheel or bristle disc with reasonably firm pressure at c.10,000 rpm.

- Can be used for polishing various prostheses (lithium disilicate, zirconia, hybrid-resin ceramics, composite)
- Polishing with Amber Glow right after the 1st contouring
- Rotate the brush and place it on the paste to apply a small amount for sufficient polishing. (It is recommended to use the enclosed wheel brush)
- It can also be used to reduce luster after glazing to seem a naturally reflective surface.



Posterior crown with Amber Mill Direct  
*Courtesy of Dr. Yao-Lin Tang*