Breeze™

SELF-ADHESIVE RESIN CEMENT

English

INSTRUCTIONS FOR USE

Français

French

MODE D'EMPLOI

Deutsch

German

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Italiano

Italian

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Español

Spanish

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Breeze™ Self-Adhesive Resin Cement

Breeze cement is a self-adhesive resin cement indicated for the cementation of virtually all types of restorations. Its high bond strength and low expansion makes *Breeze* cement the perfect choice for virtually all your cementation needs. Available in 3 shades, this new self adhesive cement is an ideal choice for esthetic composite and cementable all-ceramic restorations.

By eliminating the need for etching and bonding, *Breeze* cement reduces both sensitivity and chair time. *Breeze* cement is packaged in our popular double barrel syringe with automix tips, providing versatility, speed and accuracy during placement; no hand-mixing or trituration devices are required.

INDICATIONS:

- Cementation of composite, cementable all-ceramic, PFM and alloy crowns, bridges, inlays, and onlays.
- Cementation of fiber reinforced composite posts.
- Cementation of titanium, titanium alloy, stainless steel and zirconia posts.

CONTRAINDICATIONS:

- Cementation of porcelain veneers.
- Cementation of feldspathic porcelain restorations.
- Cementation of all-ceramic restorations contra-indicated for cementing procedures.

Storage and Handling

Storage: Refrigeration required.

Store at 35°-55° F (2°-12° C).

Use: Use at room temperature.

Preparation

1. Trial seat the restoration prior to cementation to ensure proper fit and clean thoroughly.
2. Prepare the restoration according to the manufacturer's instructions.
Note: When placing an all-ceramic restoration, apply silane to the internal surface of the restoration and air dry.
3. Prepare and clean the tooth to be restored. Rinse the prepared tooth thoroughly and remove excess water by briefly drying it with a 2 second stream of air. Do not over dry the tooth.

**Pentron
Clinical**
Technologies, LLC

68 N. Plains Industrial Rd.
Wallingford, CT 06492 USA
tel: 203.265.7397

EC/REP CEpartner4U, 3951DB; 13. NL.
tel: +31 (0)6.516.536.26

Important Note: Although there is no separate etching or bonding procedure when using *Breeze* cement, the tooth surface should be free from any contamination such as saliva, blood, eugenol, or any other substance that may interfere with the setting and/or bonding of resin adhesives.

Cementation

1. Prepare the syringe of *Breeze* cement for initial use.
 - 1a. Remove the cap from the syringe. Observe the level of cement base and catalyst in the two orifices of the syringe. If the level of material is not even, bleed excess material from the syringe until the base and catalyst both flow evenly.
 - 1b. Attach the mixing tip to the syringe.
Note: Mixing tips with intra-oral tips are enclosed, if preferred for cement application. Order Item Codes N72AA (50 pk) or N72AB (100 pk) if additional intra-oral tips are desired.
2. Gently dispense the cement directly into the restoration.
Note: If cementing a post, use a lentulo spiral to completely coat the canal walls with cement. Use of the intra-oral tips noted above is recommended.
3. Place the restoration firmly and carefully on the prepared tooth and ensure that it is fully seated within 1 – 1½ minutes from initial mixing.

Important Note: When placing multiple units, it may be necessary to replace the mixing tip during use. The working time of *Breeze* cement is 1½ minutes which begins as soon as material is first dispensed through the mixing tip. As a result, working time decreases with each unit that is placed.

4. Light cure the margins of the restoration for one second to gel the cement and facilitate clean up. Pentron Clinical recommends the Avanté™ LED Curing Light or the Avanté™ Halogen Light.

5. Remove excess cement with a scaler or explorer.

Note: *Breeze* Self-Adhesive Resin Cement will completely set in approximately 3½ - 4 minutes. If the restoration is translucent, it may be light cured for 40 seconds to achieve an immediate set.

Working Characteristics

Mode of Cure:	Dual cure
Method of Mixing:	Auto-Mix
Mixing Ratio (base:catalyst)	1:1
Mixing Time	Auto-Mix
Working Time (maximum)	1 - 1½ minutes
Intra-oral Setting Time (from initial mix)	3½ - 4 minutes

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