<b><u>Hrvatski/Croatian</u></b> . . . . .	<b>p2</b>
<b><u>Česky/Czech</u></b> . . . . .	<b>p3</b>
<b><u>Dansk/Danish</u></b> . . . . .	<b>p4</b>
<b><u>Nederlands/Dutch</u></b> . . . . .	<b>p5</b>
<b><u>English</u></b> . . . . .	<b>p6</b>
<b><u>Eesti/Estonian</u></b> . . . . .	<b>p7</b>
<b><u>Suomi/Finnish</u></b> . . . . .	<b>p8</b>
<b><u>Français/French</u></b> . . . . .	<b>p9</b>
<b><u>Deutsch/German</u></b> . . . . .	<b>p10</b>
<b><u>Elinika/Greek</u></b> . . . . .	<b>p11</b>
<b><u>Magyar/Hungarian</u></b> . . . . .	<b>p12</b>
<b><u>Íslenska/Icelandic</u></b> . . . . .	<b>p13</b>
<b><u>Italiano/Italian</u></b> . . . . .	<b>p14</b>
<b><u>Lietuviška/Lithuanian</u></b> . . . . .	<b>p15</b>
<b><u>Norsk/Norwegian</u></b> . . . . .	<b>p16</b>
<b><u>Polski/Polish</u></b> . . . . .	<b>p17</b>
<b><u>Português/Portuguese</u></b> . . . . .	<b>p18</b>
<b><u>Română/Romanian</u></b> . . . . .	<b>p19</b>
<b><u>Srpski/Serbian</u></b> . . . . .	<b>p20</b>
<b><u>Slovenčina/Slovakian</u></b> . . . . .	<b>p21</b>
<b><u>Slovenščina/Slovenian</u></b> . . . . .	<b>p22</b>
<b><u>Español/Spanish</u></b> . . . . .	<b>p23</b>
<b><u>Swenska/Swedish</u></b> . . . . .	<b>p24</b>
<b><u>Türkçe/Turkish</u></b> . . . . .	<b>p25</b>

**Correct Plus™ brzo stvrdnjavanje komplet - Nano-ojačan hidrofilni materijal za uzimanje otisaka s brzim stezanjem**

**Skladištenje i uporaba:**

*Correct Plus* Fast Set materijal za uzimanje otisaka. Čuvajte i koristite na temperaturi od 65° - 75°F (18° - 24°C).

**Indikacije**

- Uzimanje otiska za 1-3 jedinice.
- Izrada dvojnika kamenih modela.

**Kontraindikacije**

- Zbog brzog stezanja *Correct Plus* Fast Set se ne preporuča za uzimanje otisaka lukova. Za lukove koristite *Correct Plus* standardni komplet.

**Valne napomene**

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## INSTRUCTIONS FOR USE:

### Correct Plus® Fast Set Hydrophilic Impression Material

#### Storage and Use:

Store and use *Correct Plus* Fast Set Impression Material at 65° - 75°F (18° - 24°C).

#### Indications

- Impressions for 1-3 units.
- Duplication of stone models.

#### Contraindication

- Due to the fast working time *Correct Plus* Fast Set is not recommended for full arch impressions. For full arch impressions use *Correct Plus* Regular Set Impression Material.

#### Important Notes

- Hemostatic agents and retraction cords that contain ferric sulfate may inhibit the set of impression material. Before taking the impression, rinse the preparation site thoroughly with water to remove any residual contaminants.
- Models may be poured 15 minutes after the impression is removed from the mouth.
- Due to its rigidity, use caution when using *Correct Plus* Fast Set Auto-Mix Putty on periodontally involved teeth or on bridges unless severe undercuts below the pontics are blocked out.

#### Product Descriptions:

- Auto-Mix Putty – a heavy viscosity material with a high end hardness designed for use in stock, custom, or triple trays.
- Universal™ Body – a medium viscosity material designed to be used as a tray material or by itself in a monophasic impression technique.
- Thick n' Thin® Light Body – a non-slumping, light viscosity material that initially stacks on the tooth then flows into the sulcus to capture subgingival detail.

#### Instructions for Taking Impressions

1. Apply Correct VPS® Tray Adhesive to the tray and allow to air dry for 5 minutes.
2. Load Auto-Mix Putty, *Universal* Body, and/or *Thick n' Thin* Light Body cartridge(s) into a dispenser gun(s).
3. Remove caps and extrude a 1/4 inch of material to ensure equal amounts of material are dispensed.
4. Attach mixing tips to the cartridges. Attach an intraoral tip to the wash material mixing tip (*Thick n' Thin* Light Body or *Universal* Body.)
5. Dispense either *Correct Plus* Fast Set Auto-Mix Putty or *Universal* Body into the tray while simultaneously syringing either *Thick n' Thin* Light Body or *Universal* Body around the preparation(s). Working time is 60 seconds.
6. Seat the tray.
7. Retain the impression in position for 2 minutes.
8. Remove the impression from the mouth.
9. Rinse thoroughly under water and disinfect.
10. If pouring a model, wait 15 minutes before proceeding.

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