

brilliance!

Professional TAKE-HOME Whitening System



brilliance!

INSTRUCTIONS



1-800-667-9622
www.synca.com

337 Marion, Le Gardeur, QC, Canada, J5Z 4W8



brilliance!

INGREDIENTS:

Propylene Glycol, Kosher Glycerine, Carbamide Peroxide (22%), Potassium Nitrate, Carbomer, Peppermint Essential Oil, Triethanolamine, Xylitol.

CAUTION:

For cosmetic use only. Not recommended if patient is pregnant, nursing or is under 14 years of age. Do not use if patient presents allergies to any of the ingredients. If in contact with eyes, rinse immediately. Store in a cool place.

INSTRUCTIONS

Brush and floss. Place one drop of gel in the custom Brilliance tray adjacent to each tooth that will be whitened. Using both hands, delicately insert the tray into your mouth. Expectorate the surplus gel as it may cause throat irritation.

The safest whitening method is to wear the Brilliance tray 30-45 minutes, or by following the specific instructions of your dentist.

Prolonged wearing of the tray is indicated only when recommended by your dentist and if lesser times have shown to be ineffective.

If you experience sensitivity, ask your dentist for a stannous fluoride gel. This gel can be brushed on your teeth once or twice daily and can reduce sensitivity considerably.

To obtain the most benefit from this treatment, you must maintain excellent oral hygiene with brushing and flossing.

Always remove the Brilliance tray when eating. Brush both the inside and outside of the tray daily.

When commencing treatment, your dentist should examine your mouth every week or two to check for evidence of irritation to teeth, gums or dental restorations.

If you have white filling or porcelain restorations, your teeth may become significantly lighter than these restorations.

It is highly recommended that you refrain from smoking during treatment as carbamide peroxide may increase the carcinogenic effect of tobacco.

If you experience discomfort or problems of any nature, consult your dentist immediately for instructions.